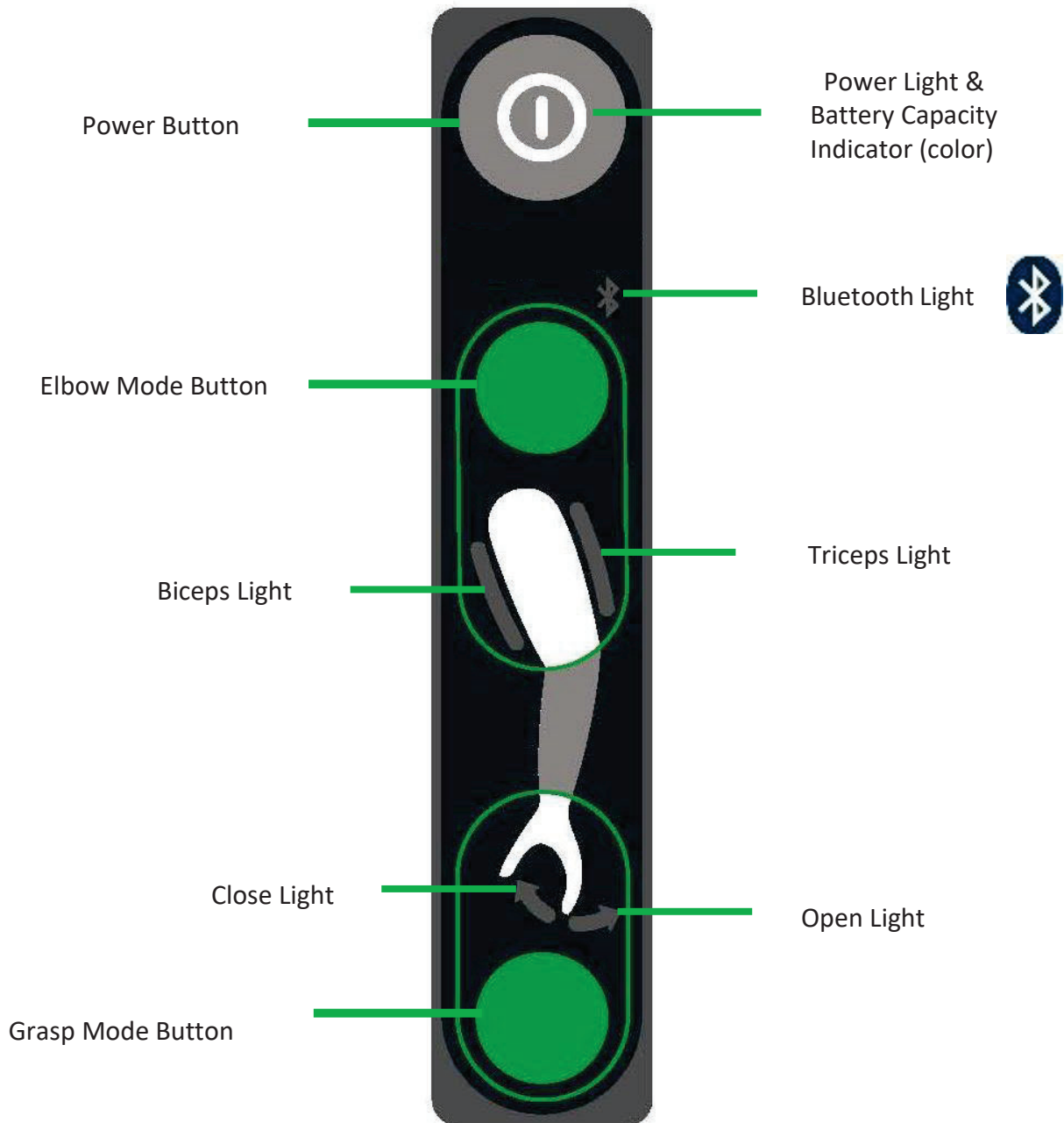

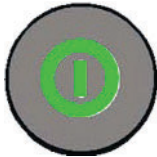
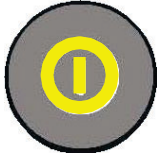
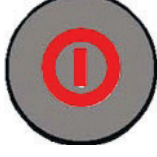





## CONTROL PANEL BUTTONS & LIGHTS DIAGRAM



# CONTROL PANEL LIGHTS, DESCRIBED

## POWER LIGHT & BATTERY CAPACITY INDICATOR COLOR

NOT Illuminated (off)		Clamp Champ is powered OFF.
Illuminated Green		Clamp Champ is powered ON. The battery has greater than 15% charge remaining.
Illuminated Yellow		Clamp Champ is powered ON. The battery has between 15% and 5% charge remaining.
Illuminated Red		Clamp Champ is powered ON. The battery has less than 5% charge remaining.
If color is changing in a repetitive pattern	<ol style="list-style-type: none"> <li>1. White (may look light-blue)</li> <li>2. Dark Blue</li> <li>3. Green</li> <li>4. Red</li> </ol>	<u>Clamp Champ is experiencing an error. See Page 25 for instructions.</u>
<p>⌚ <b>NOTE:</b> Another way to check the battery capacity level is to eject the battery and check the light indicator on the battery itself (see Page 25 for instructions).</p>		

<b>BLUETOOTH LIGHT</b>		
NOT Illuminated (off)		Clamp Champ <u>is not</u> connected to Config.
Illuminated (on)		Clamp Champ 2x <u>is</u> connect to Config.
Flashing (blinking)		<u>Clamp Champ is experiencing an Error. See Page 36 for instructions.</u>

## HOW TO POWER THE CLAMP CHAMP ON AND OFF

**⚠ WARNING:** Do not use the Clamp Champ to drive an automobile or operate machinery.

**⚠ WARNING:** The Clamp Champ is not waterproof. Avoid showering, swimming, or exposing the device to rain or other moisture. Do not wear the device while washing dishes by hand, as the grasp motor is at high risk of water damage.

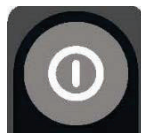
**⚠ WARNING:** Do not attempt to lift heavy objects with the Clamp Champ; the elbow motor provides at most 5 lbs of lifting assistance.

**⚠ CAUTION:** The Clamp Champ is not suitable for use in the presence of flammable anesthetic mixtures with air, or flammable anesthetic mixtures containing oxygen or nitrous oxide.

### POWER ON

➞ *Important: Only power on the Clamp Champ after it is securely and comfortably positioned on your arm. Keep the Clamp Champ powered OFF while putting it on and when removing it. Refer to the User Manual Appendix for additional instructions.*

1. Press the Power Button.



2. Observe: When the Clamp Champ is turning ON, all the lights on the control panel will light up.
3. Observe: After 3 seconds, all the lights will turn off, except for Power Button light.
4. The Clamp Champ is ready to use!

## POWER OFF

1. Press the power button.



2. Observe: All the lights will stop being illuminated, indicating that the Clamp Champ has successfully powered off.

⊞ **NOTE:** The Clamp Champ should remain OFF any time it's not being worn.