



ARMOURBITE® / ARMOURSHIELD® MOUTHGUARD



MOUTHGUARD

READ ALL INSTRUCTIONS PRIOR TO FITTING YOUR MOUTHGUARD.

What you need for stove-top fitting:

- Pot for boiling water
- Bowl of cold water
- Mirror
- Timer



1. Bring pot with 4 inches of water to a boil and then remove from heat source. **



2. For fitting purposes, attach strap to mouthguard, and while holding strap, submerge mouthguard in hot water for 90 seconds.



3. Immediately submerge mouthguard in a bowl of cold water for no more than 2 seconds.



4. To ensure proper fit, use a mirror to visually align mouthguard to wrap around your teeth, and simultaneously perform below steps for 1 minute:

- Bite down gently and naturally on mouthguard.
- Press tongue firmly against the roof of mouth and with light sucking action, create a vacuum chamber by drawing out all water and air.
- Press fingers along outside of mouthguard to form to teeth.




5. After fitting process is complete, remove mouthguard from mouth and place in bowl of cold water for 5 minutes to set the soft material.

After fitting mouthguard, the strap can be replaced with the ArmourPlate. These are interchangeable by twisting the mouthguard, causing the plate or strap to pop out of the mouthguard.

ArmourBite®/ArmourShield® Mouthguards are designed to be worn on the top teeth and must be fitted as instructed to function properly. A damaged, altered or improperly fitted mouthguard will not provide the same protection.

ARMOURBITE® MOUTHPIECE

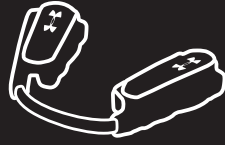

MOUTHPIECE

What you need for stove-top fitting:

- Pot for boiling water
- Bowl of cold water
- Timer
- Slotted spoon
- Mirror



1. Bring pot with 4 inches of water to a boil and then remove from heat source. **



2. Submerge mouthpiece with the logo facing up in hot water for 45 seconds.



3. Remove mouthpiece with slotted spoon, and allow to cool for 15 seconds.



4. Place the mouthpiece in your mouth. While holding the labial bar tight to your lower front teeth with your middle fingers, use your index fingers to press the mouthpiece down and onto your lower teeth. Perform below steps for 1 minute:

- Bite down naturally.
- Gently pack the mouthpiece against your gums by pressing on the outside of your cheeks, or directly on the mouthpiece.
- Gently suck against the mouthpiece.



5. After fitting process is complete, remove mouthpiece from mouth and place in a bowl of cold water for 10 minutes to set the soft material.

ArmourBite® Mouthpiece is designed to be worn on the lower teeth and must be fitted as instructed to function properly.

CARE INSTRUCTIONS

- Do not trim your mouthguard. Trimming will cause the materials to separate and void your warranty.
- Do not leave your mouthguard in direct sunlight or in a hot car for extended periods of time.
- UA Mouthwear Freshener spray is recommended after each use to keep your mouthguard fresh and clean.
- Protect your mouthguard by storing it in the UA Mouthwear case.



PRODUCTS AVAILABLE AT ARMOURBITE.COM.

ARMOURBITE®/ARMOURSHIELD® MOUTHGUARD DENTAL LIMITED WARRANTY

CERTIFICATE OF LIMITED WARRANTY AND DISCLAIMER

WARRANTY: Bite Tech, Inc. (Bite Tech) warrants this mouthguard to be free from defects in material and workmanship which could cause damage or injury to the sound, natural teeth of the purchaser herein when properly fitted and used for a period of one (1) year from the date of purchase.

LIMITATION OF LIABILITY: This limited warranty applies only to injuries to the purchaser's teeth that are free from any pre-existing natural or unnatural defect, including prior dental work. This limited warranty only applies if the purchaser suffered a dental injury in a supervised athletic contest or coach-supervised training session not otherwise excluded. No warranty is provided for any damage or injury incurred during or in connection with any martial arts, mixed martial arts, boxing or similar fighting or combat sport or activity. Bite Tech's obligation under this warranty shall be limited to the actual costs incurred by the user for necessary dental treatment by an appropriately licensed dental professional up to a maximum liability of \$500 per injured or replaced tooth with a maximum liability per incident of \$32,000. Bite Tech's obligation shall be reduced by any insurance coverage available to the user and applicable to the treatment of the injury claim. This limited warranty does not provide payment or indemnity for orthodontic treatments or preparation of a tooth to be used as an anchor for a bridge. The remedies set forth herein are exclusive and the total liability of Bite Tech with respect to use of this mouthguard, whether based on contract, warranty, negligence, indemnification, strict liability or otherwise. In no event shall Bite Tech be liable for consequential, incidental or special damages.

CONDITIONS: This limited warranty applies only to injuries to the purchaser's teeth that are free from any preexisting natural or unnatural defect, including prior dental work. This limited warranty only applies if the purchaser suffered a dental injury in a supervised athletic contest or coach-supervised training session. Bite Tech shall have no responsibility or liability hereunder, unless:

- The mouthguard was fitted in accordance with the printed instructions that accompany the mouthguard; and
- The mouthguard was not damaged or altered in any way, whether by chewing on the mouthguard or by other action to alter its configuration or chemistry (other than by the prescribed fitting), except as occurs by natural use; and
- a complete, written notice of the injury and damage, together with a copy of the receipt of purchase, is received by Bite Tech within thirty (30) days after the occurrence of the injury; and
- the user provides a written statement from an appropriately licensed dentist providing necessary treatment within thirty (30) days after the date of injury that substantiates the injury, damage and treatment provided, together with identification of insurance coverage and insurance payments; and
- the user provides a statement by a supervising coach or participating game official that the mouthguard was in proper use at the time of injury; and
- the applicable mouthguard is returned to Bite Tech with the written notice of injury. The mouthguard shall become the property of Bite Tech upon receipt of return.

NO MOUTHGUARD CAN PREVENT ALL INJURIES OR ALL DENTAL DAMAGE AND NO SUCH WARRANTY IS BEING MADE. THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES FOR A PARTICULAR PURPOSE AND ANY IMPLIED WARRANTY ARISING OUT OF A COURSE OF A DEALING, CUSTOM, USAGE OR TRADE. EXCEPT AS IS OTHERWISE EXPRESSLY PROVIDED HEREIN, BITE TECH DISCLAIMS ALL LIABILITY FOR LOSS, INJURY OR DAMAGE BASED UPON NEGLIGENT DESIGN OR DEFECTS IN MATERIALS AND WORKMANSHIP FOR THE WITHIN MOUTHGUARD.

Keep this dental warranty, disclaimer and store receipt in your records. In case of injury, mail mouthguard, store receipt, the notice of injury and accompanying statements to Bite Tech, Inc. This product is protected by one or more U.S. and international patents and pending patent applications.

ARMOURBITE® MOUTHPIECE PRODUCT REPLACEMENT WARRANTY

Bite Tech warrants this mouthpiece to be free from defects in material and workmanship which could cause damage or injury to the sound, natural teeth of purchaser herein when properly fitted and used for a period of one (1) year from the date of purchase. This warranty is made in lieu of all other warranties, express or implied, including but not limited to the implied warranty for particular purpose and any implied warranty arising out of a course of dealing, custom, usage or trade. Except as is otherwise expressly provided herein, Bite Tech disclaims all liability for loss, injury or damage based upon negligent design or defects in materials and workmanship in the mouthpiece. Product which has been abused or that has exceeded its useful lifetime through normal use may not be returned. Abuse may include but is not limited to chewing, trimming, and/or using the product in a manner that is inconsistent with its intended use.

For all warranty claims related to self-fit Under Armour Performance Mouthwear™ products, please contact BITE TECH customer service at customerservice@bitetech.com or call 1-877-248-3832 for more information.

** CAUTION: BOILING WATER AND ITEMS THAT HAVE BEEN MICROWAVED WILL BE HOT AND CAN CAUSE INJURY.

Bite Tech • 20 Glover Avenue • Norwalk, CT 06850 • UA approved licensee • Visit www.bitetech.com

MAN-01027-001